



Town of Dennis Surf/Mobi Beach Chair Rental

The Town of Dennis has handicap accessible beach chairs available to be signed out at the following beaches:

Corporation, Mayflower and West Dennis Beach

These chairs, with their over sized tires, make over the sand travel possible for those who require wheelchairs.

To reserve a chair at Mayflower or Corporation, please call 508-385-1558, seven days a week, 8 a.m. to 5 p.m. To reserve a chair for West Dennis Beach, please call 508-760-6240, seven days a week, 8 a.m. to 5 p.m.



Mobi Chair



Surf Chair



Town of Dennis Surf/Mobi Chair Waiver of Liability

IN CONSIDERATION OF YOUR ACCEPTING MY WAIVER, I, INTENDING TO BE LEGALLY BOUND, DO HEREBY FOR MYSELF, MY HEIRS, EXECUTORS, AND ADMINSTRATORS, WAIVE AND RELAEASE FOREVER, ANY AND ALL RIGHTS AND CLAIMS OR DAMAGES I MAY ACCRUE AGAINST THE TOWN OF DENNIS, THEIR SUCCESSORS, REPRESENTATIVES AND ASSIGNS, FOR ANY AND ALL INJURIES SUFFERED BY ME WHILE USING THE SURF CHAIR. WARNING AND CONCERNS ABOUT THE USE OF THE SURF CHAIR HAVE BEEN PROVIDED TO ME.

THE UNDERSIGNED HAS READ ALL OF THE INSTRUCTIONS AND UNDERSTANDS THAT IT IS THEIR RESPONSIBILITY TO FOLLOW THE MANUFACTUERS SAFETY INSTRUCTIONS.

Signature of User/Legal Guardian

Date

If user is younger than 18, must be signed by Legal Guardian.

Name

Telephone #

Address

Town

State

Zip Code

Beach Name

ID collateral

WARNING

1. Failure to have the assisting individual present while chair is in the water, could result in serious injury or death by drowning.
2. Assisting individual/or user should not undertake task beyond his/her physical ability (ramp, deep sand).
3. Do not take a Surf Chair in water over 6 inches. Chair will become positively buoyant and tipsy. The Mobi-Chair can be taken into the water.
4. Use of sunscreen and/or umbrella is recommended to avoid serious sunburn.
5. The assisting individual must never leave chair unattended when occupied especially in or around water because of tidal, sun/or weather changes.
6. Every assisting individual should take notice of where lifeguard stations are located.
7. Chairs must be returned at least 30 minutes before the close the beach so chair may be washed down and stored for the night.
8. If there is more than one request for the chair, use will be limited to a 2-hour period.
9. Turning of the chair is accomplished by tilting chair backward and pivoting.
10. Remember the safety of the person in the surf chair is dependent on the ability of the assisting individual.

MOBI-CHAIR OPERATING INFORMATION

Operating the wheelchair

- The wheelchair is well suited for normal weather conditions and firm surfaces. For safety purchase, please avoid using the wheelchair in adverse weather and other poor conditions.

User Indications

- Operating the wheelchair may injure user or damage the wheelchair if the user is unable to have full control. If the user is not mentally or physically able to keep full control of the wheelchair at all times, an assisting individual must be present when necessary.

Safety and Operating Information Stability

- This wheelchair has been designed to provide the stability needed during normal daily activities. Therefore, proper operation of the wheelchair will maintain balance at all times. The impact on the center of gravity is under any wheelchair movement. A change in the center of gravity may lead to the wheelchair tipping over and a fall. When you transfer your weight from one place to another, we recommend using seat belts to improve your safety.

Weight Limit

- The maximum limitation weight of the user is 330 lbs. / 150 kg.

Caution

- Do not use parts that can be detached, such as armrests, for support or any other reasons other than the intended purposes. Do not use them as a means for lifting the product, as they may inadvertently release. This may result in injury to the user and/or the assisting individual.
- Avoid turning the chair suddenly and never try to climb or descend a ramp diagonally.
- Although the Mobi-Chair can float on water, you cannot use or treat it as a boat. In normal conditions of use, the Mobi-Chair will not turn over if the user sits sturdily. However, for safety and requirement by state law, a life jacket is to be worn by the user. The Mobi-Chair is not intended for areas with strong currents and waves.
- Please be sure to use Mobi-Chair in a range in which the lifeguard can reach timely.
- Be careful and have fun!



Lifeguard Instruction for Surf/Mobi Chair

1. **Check-Out Procedure:**
 - a. Be sure Surf/Mobi Chair is signed out by an adult.
 - b. The adult must leave a license or ID for collateral.
 - c. Place the ID in the binder front pocket.
 - d. Ensure individual signs Waiver of Liability.
 - e. Provide individual with a Mobi-Chair Operating Information sheet.
 - f. Record name, date, and beach in the check-out chart in the front of the binder.

2. **Be sure adult accepting responsibility for the Surf/Mobi Chair signs the “Waiver of Liability” and understands the following:**
 - a. They should not undertake task beyond his/her physical ability (ramp, deep sand).
 - b. They should not take Surf chairs in water over 6” in depth. Chair will become positively buoyant and tipsy. The Mobi chairs may be taken into the water.
 - c. Never leave chair unattended when occupied, especially in or around the water because of tidal, surf or weather changes. Failure to have an attendant present while chair is in the water could result in serious injury or death by drowning.
 - d. Use of sunscreen or umbrella is recommended to avoid serious sunburn.

3. Be sure to review location of lifeguard stations for that day.

4. Remind users of beach hours. Have chair back to the beach office 30 minutes before close of beach so it may be washed down and put away.

5. If there is more than one request for the chair, use will be limited to 2-hour period.

6. The safety of the person in the chair is dependent on the ability of the assisting individual.

7. Please notify the assisting individual and/or user it is their responsibility to supply the user with a personal flotation device (life jacket) as stated in Massachusetts state laws and regulations.

8. **Return Procedure:**
 - a. Return the license or ID to the individual.
 - b. Record on the check-out chart that the chair has been returned.
 - c. The chair needs to be sprayed with a disinfectant and left out to air dry. Record on the check-out chart that the chair has been cleaned.